

# Proctowash™


## Rectal Washout Kit



### Description

Developed in close association with colorectal surgeons and OR nurses, the **Proctowash™** is a quick, sealed, easy-to-use kit for washing out the rectum and distal colon. The device comprises:

- A** Probe with flexible tip for atraumatic insertion without obturator; sealed inlet port for irrigation, and sealed outlet port for waste/drainage
- B** 2.5m fluid administration set with atraumatic tip
- C** Large bore waste tubing
- D** Detachable, resealable 3-litre container
- E** Sealing cap

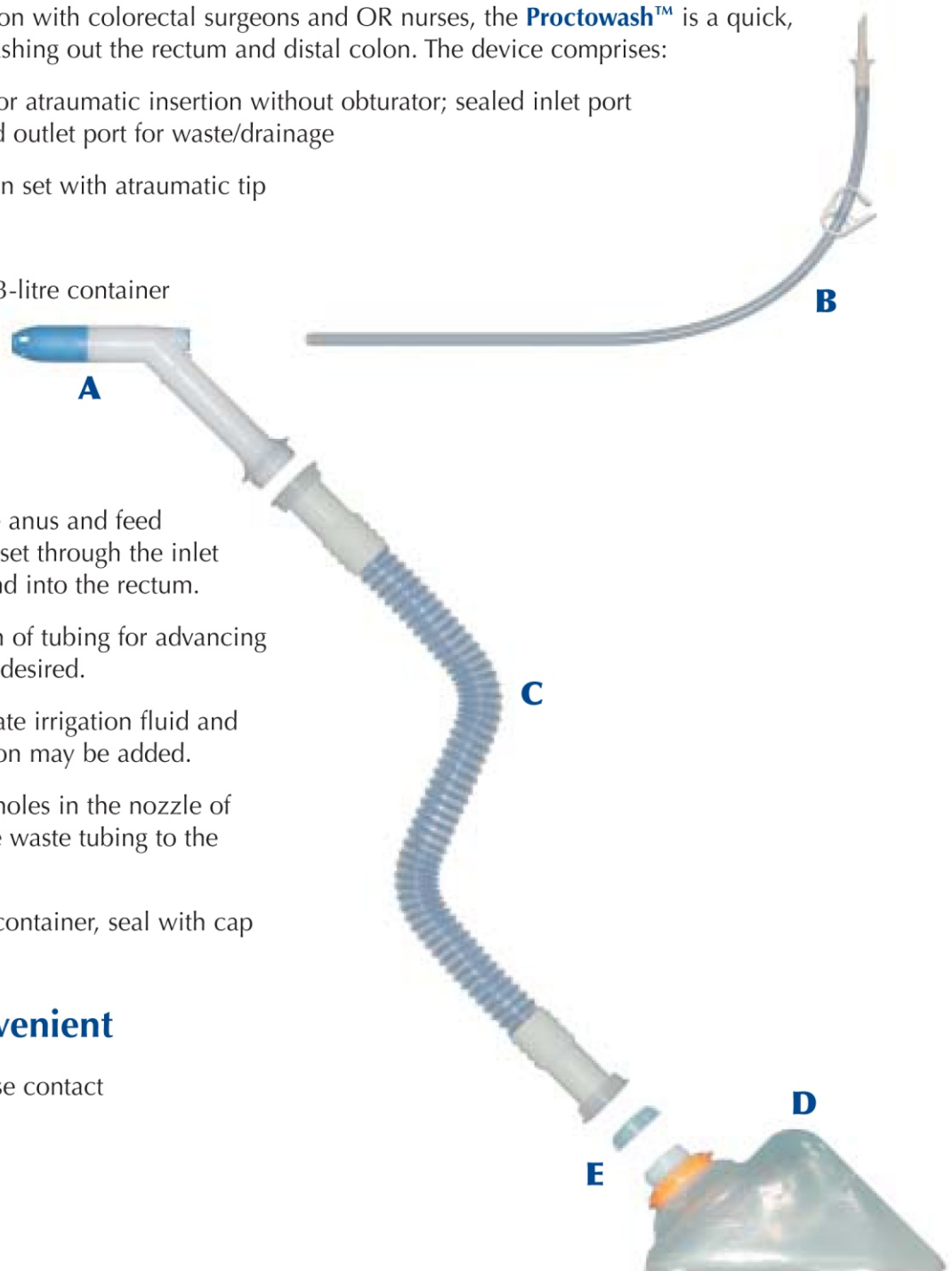

### How to Use

- 1** Insert the probe into the anus and feed the fluid administration set through the inlet port, along the probe and into the rectum.  
There is sufficient length of tubing for advancing further into the colon if desired.
- 2** Spike a bag of appropriate irrigation fluid and irrigate. A topical solution may be added.
- 3** Effluent drains through holes in the nozzle of the probe and down the waste tubing to the container.
- 4** When finished, detach container, seal with cap provided and discard.

### Quick, Clean, Convenient

For further information, please contact Intermark on 877 332 9200.



**Order Code: Proctowash PR 01**

**Patents:** GB2375964, US6761702 others pending

### MAHE International

468 Craighead Street	Toll Free:	800 294 7946
Nashville, TN 37204	Tel:	615 269 7256
United States	Fax:	615 269 4605
	e-mail:	info@maheinternational.com

[www.maheinternational.com](http://www.maheinternational.com)