

On-Table Lavage Kit

Description

The Retrowash is a quick easy, all-in-one device for on-table lavage (intra-operative colonic lavage) in colorectal surgery. It effects a rapid, clean and odourless washout, facilitating primary anastomosis. It also enables on-table colonoscopy following lavage.

The device attaches to the bowel proximal to the lesion, and uses a very effective new retrograde irrigation technique (see below). Antegrade irrigation may also be used.



Advantages

- 10-15 minutes average washout time
- No second incision required at appendix
- No leakage or soiling
- Sealed collection unit
- Enables on-table colonoscopy
- Two sizes (35mm or 46mm diameter)

Retrograde Irrigation Technique

1. Following attachment, an extension catheter is passed through a seal and along the Retrowash handpiece into the colon.
2. A high flow rate fluid administration set is connected to the extension catheter, creating a jet of irrigation to direct against the bowel contents. Effluent is washed down through the handpiece into the sealed waste receptacle.
3. As the fecal matter clears, the extension catheter is advanced further along the colon toward the cecum, irrigating in 250ml bursts.

CE 0120



Order Info: 35mm Retrowash RW35
46mm Retrowash RW46

Patents: GB2375961, GB2375964, US6761702

MAHE International

468 Craighead Street
Nashville, TN 37204
United States

Toll Free: 800 294 7946
Tel: 615 269 7256
Fax: 615 269 4605
e-mail: info@maheinternational.com

www.maheinternational.com